


CICAVET'EAU

Technical data sheet

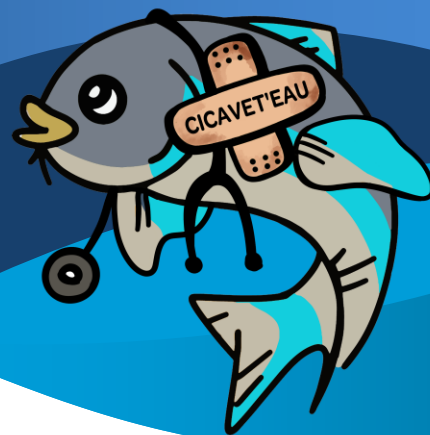

COMPOSITION

- Povidone-iodine
- Gelling powder

AVAILABLE FORMATS

- 10g
- 30g

PRODUCT DESCRIPTION AND PROPERTIES

Cicavet'eau is an **external dressing** specifically designed to **support the natural regeneration of the skin** of fish and other aquatic animals.

Thanks to its **powder formulation** that **gels upon contact with water**, Cicavet'eau effectively **isolates the affected area** from the **external environment** while keeping the powder in place.

Its formulation helps maintain a **clean and protective environment**, favourable to **natural tissue regeneration**.

The product is **painless, non-irritant, alcohol-free**, and contains **no aggressive excipients**. Once gelled, Cicavet'eau forms a **lightly adherent surface layer**, preventing contamination by environmental impurities and limiting product loss through friction.

Cicavet'eau is designed for use on **all external parts of the fish's body**.
It is **not recommended for use on gills**.

Depending on the application area and specific conditions, the dressing may remain in place for **one to several hours**.

This product is **manufactured in France** by Vet'eau Veterinary Practice.
No prescription required.

⚠ **Do not apply on gills** ⚠



FABRIQUE PAR :
Vet'eau

1289 Rue des Pyrénées – 31300 Grenade
Tel : +33 (0)5 62 79 07 80
www.veteau.fr



DIRECTIONS FOR USE

1 Remove the animal from the water.

Gently pat the lesion and surrounding area with a compress to obtain a moist (not wet) surface. This improves the adhesion of Cicavet'eau.

2 Apply a thin layer of Cicavet'eau powder on and around the affected area.

Spread the powder slightly beyond the wound edges to enhance adhesion and protect the surrounding skin.

3 Immerse the animal in water to activate gel formation.


After application, gently return the fish to the water to trigger the gelling process.

Avoid applying a thick layer, which could reduce adhesion.

For multiple small lesions, a fine water mist spray can be used to initiate gelling while keeping the fish out of the water.

RECOMMENDATIONS

- For **older or irregular skin areas**, once the surface has been cleaned and damaged tissue removed, Cicavet'eau can be applied as a **protective dressing**, alone or in combination with another **local care product**.
- ⚠ **Important:** This product **does not replace appropriate veterinary supervision**. Always consult an **aquatic veterinarian or fish health specialist** in case of persistent or ulcerative lesions.

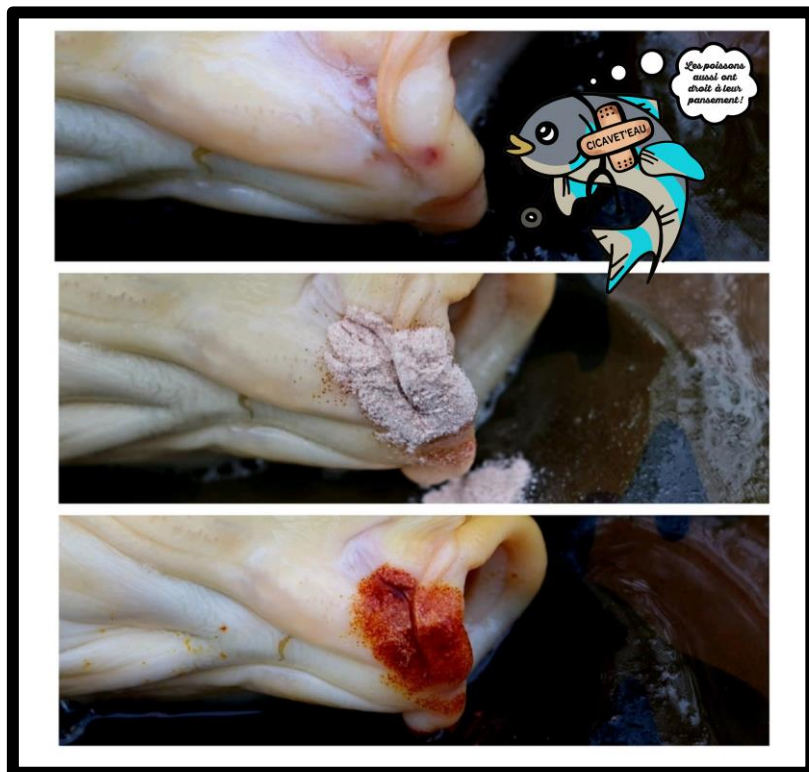

FABRIQUE PAR :

Vet'eau

1289 Rue des Pyrénées – 31300 Grenade

Tel : +33 (0)5 62 79 07 80

www.veteau.fr

